


Application

Fixation for nasal sondes

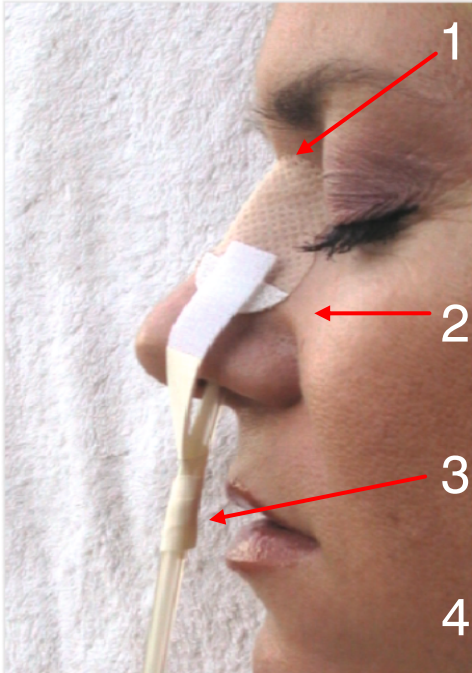

1 Nasal holder

Prepare the nose dry and free of Grease.

Important: stick the holder on the upper part of the nose, the tip is to soft and the holder comes off.

2 Nasal band


Pull the band off the foil and attach it on one side of the holder to the Velcro.

3

Wrap the adhesive strip of the band a few times around the sonda.

4

Stick the end of the strip on it self, ist then more easy to get it off.

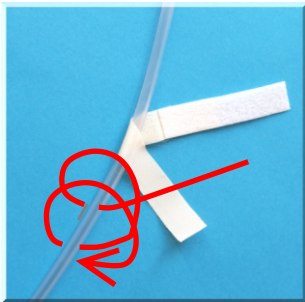

To **clean the nose**, simply loose the ribbon. The holder remains on the nose.

Hydrocolloid! Clean nose only with water, form holder well on the nose.

Nasal - Set

Nasal-set
ord.no 8500400 **Hydrocolloid**

Nasal-set
ord.no 8500401 Nonwoven

Nasal holder
ord.no. 8500410 Nonwoven

Nasal band
ord.no. 8500440



Depending on the application, it lasts up to 3 days, Hydroc. 5 days

Most parts of the SECUTAPE® system can be combined with each other, Secutape fixations are reliable and offer the best possible hold on the skin. They can be used for several days.