



Application

Tubus-Band SILICON

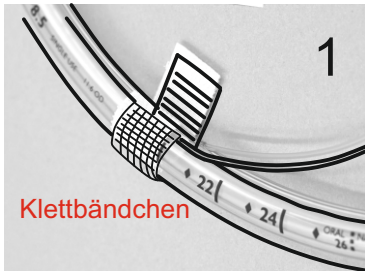


Peel off the cover of the Velcro strip and wrap it **overlapping**, around the cleaned, dry tube.
Du not cut the strip!

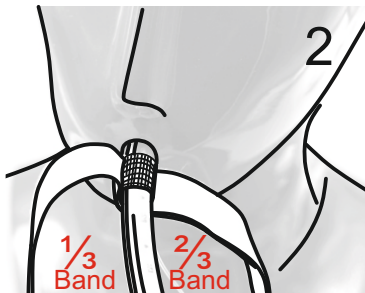

Wrap 1/3 of the band once around the velcro surface on the tube.
The other part wind around the head, e.g. over the ears and **turn the side with the silicone to the skin!**

Cut off excess length.

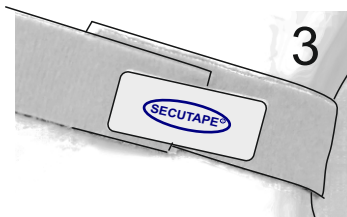

Close with the SECUTAPE tag.
Dont tie the band to tight, it does not stretch, otherwise you could damage the skin !!

The silicone-coated nylon band does not slip on the face and offers a high level of security.
The band is hypoallergenic, **does not stretch and does not absorb liquid.**
It is a safe fixation for endotracheal tubes, hygienic and easy to handle.
It is also suitable for drains such as urine catheters.
The special material prevents skin problems, it does not have to be changed often.

Tubus-Band SILICON

Tubus-Band SILICON
ord.no.
8500850



The Velcro strip **must be covered** with the tape, otherwise there is a risk of skin damage.



Depending on the application, it lasts up to 3 days,

Most parts of the SECUTAPE® system can be combined with each other, Secutape fixations are reliable and offer the best possible hold on the skin. They can be used for several days.