



Fixing system


Application

Universal fixation for tubes and lines especially CVC




1

Remove the backing paper and stick the holder on a suitable place.

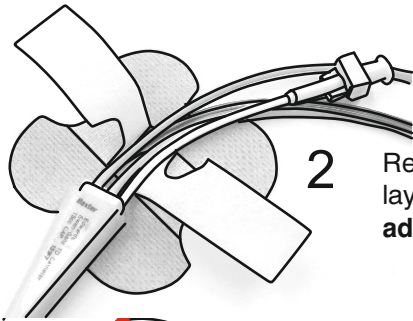

2

Remove cover from Velcro strip, lay "tubes" / lines into the adhesive area .

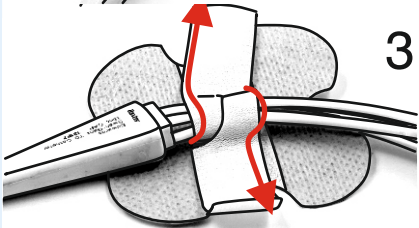

3

Close the wings and press lightly.



For fastening flat filters and other components- Simply press the filter onto the adhesive surface.

The patient can be washed or showered, the fixation **holds even if it gets wet.**
The fixation can be opened and closed several times.

Unifix



ord. no. 8501500

Unifix Hydrocolloid



ord.no.8501509



Most parts of the SECUTAPE® system can be combined with each other, Secutate fixations are reliable and offer the best possible hold on the skin. They can be used for several days.



TechniMed AG
SWITZERLAND
since 1986

Kirchstrasse 54
9400 Rorschach
info@technimed.ch

Switzerland
fon 0041/71 844 49 20
fax 0041/71 845 52 45

Germany
fon 0800 007 98 92
fax 0800 941 30 12

Austria
fon 01/440 79 79
fax 01/440 79 90

CE
Made in Germany
© TechniMed AG

www.secutate.ch