



Application

Tracheofix

Fixation for tracheal cannulas

Newly developed soft and thick nylon band.
Hypoallergenic and pleasantly soft.
It does not stretch and does not absorb liquid.

Put the band around the neck, with the **white side to the skin**.


Pull both ends through the holes of the cannula and fix them with the velcrostrap. See picture

Since the band does not stretch, **it does not need to be applied tight**.

For long term fixation or patients with skin problems. Use the additional Superabsorber pad.

The medical grade absorber will keep the skin dry and prevents skin damage.


Pull the liner of the pad and stick it inside the band. See picture.

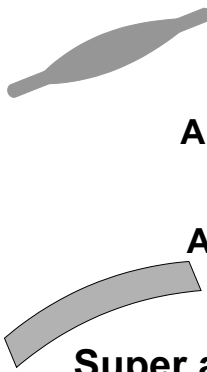

**Tracheofix
LARGE**
Art. Nr. 8501700

SMALL
Art. Nr. 8501720

Super absorber Pad
Art. Nr. 8501770

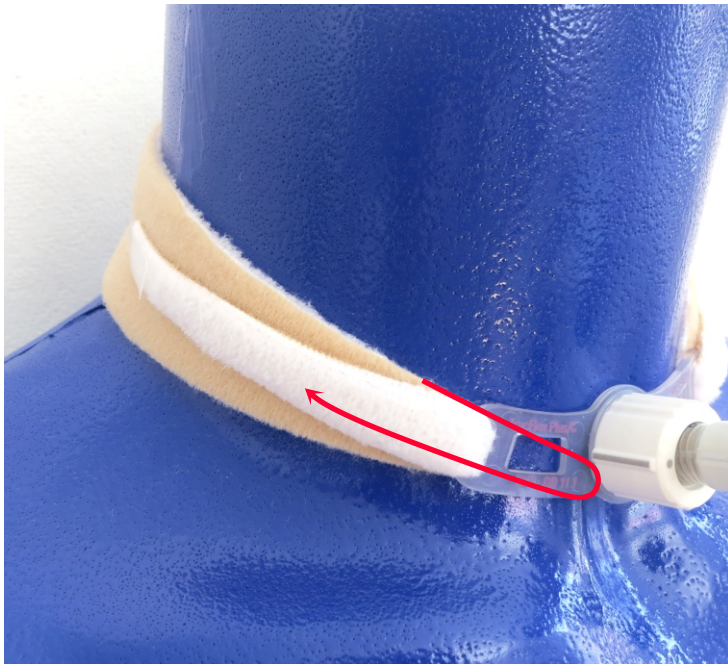

There is a possibility to cut the band and adjust it with the SECUTAPE label. See picture.



SECUTAPE fixations can usually remain on the patient for a few days and offer the best possible fixation under the conditions !!



TechniMed AG

SWITZERLAND since 1987

fon 0041 / 71 844 49 20

Kirchstrasse 54

CH-9400 Rorschach

fax 0041 / 71 845 52 45

Germany

fon 0800 007 9892

fax 0800 941 3012

Austria

fon 01/ 440 7979

fax 01/ 440 7990



Made in Germany

www.secutape.ch