


Application

SECUTAPE® fixing set for large tubes

Especially for thoracic drains and urine catheter.

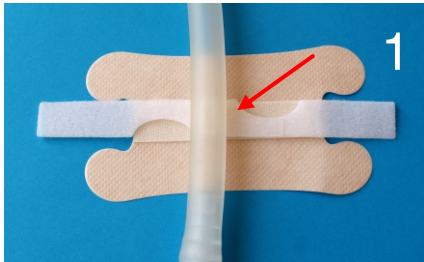


Stick the holder to the clean skin, take the liner off the strap and put the tube on the adhesive line.


Lay both ends over the tube and close the velcrostrap (s. picture)

Hydrocolloid

Take the back cover **slowly off!!**
Cleanse skin only with water and soap!

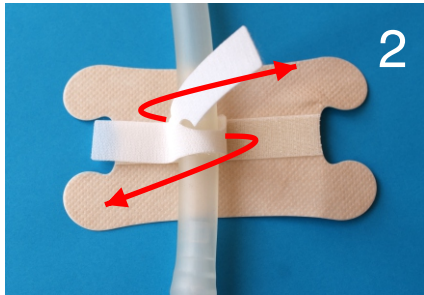

Lay both ends of the strap over the tube, *see picture.*

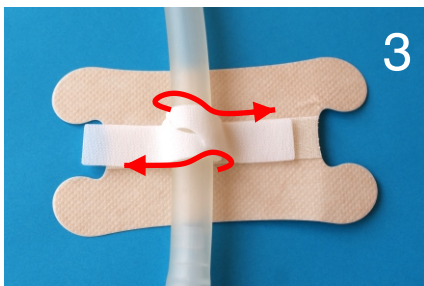

Close the straps and press lightly down, dont cover all of the Velcro. Its more easy to open!

Fixation can be opened and closed several times.


The fixation lasts even if it gets wet
Hydrocolloid, clean the skin only with water,
mold well after application!

Fixing-set for large tubes

Fixing-set nonwoven
ord.no. 8501201



Fixing-set **Hydrocolloid**
ord.no. 8501200

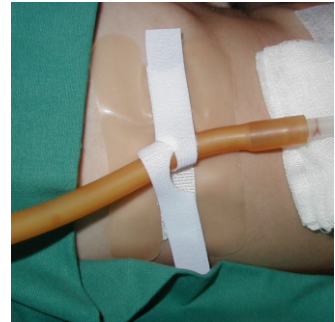

Depending on the application,
it lasts up to 3 days,
Hydroc. 6 days

Most parts of the SECUTAPE® system can be combined with each other,
Secutate fixations are reliable and offer the best possible hold on the skin.
They can be used for several days.