
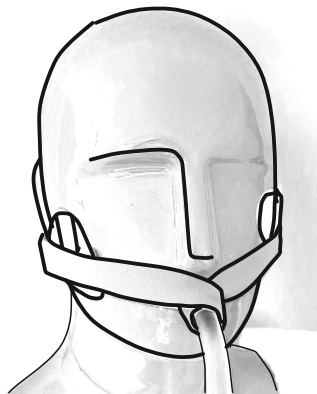
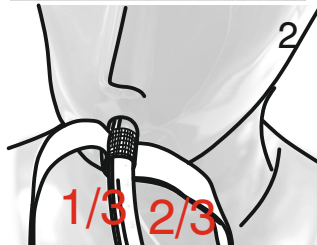


Application


Universal-Band

Fixing band for all tubes and lines especially for endotracheal tubes & tracheal cannulas.

Endotracheal tubes

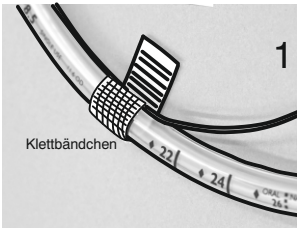

Tracheal cannulas

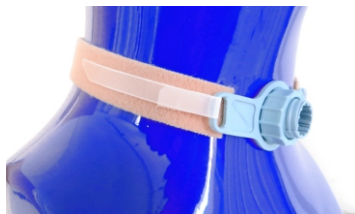


Do not stretch too tightly, the strap does not stretch.

Used material:

band, Velcro strap, SECUTAPE pad.

- take the cover of the Velcro pad and wind it overlapping around the cleaned and dry tube
- dont cut it!
- Wrap 1/3, 2/3 of the band around the tube on the Velcro pad..
- Place the band around the neck and close it with the SECUTAPE pad.
- If too long just cut it off..

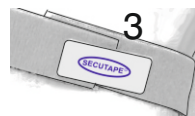

For very restless patients

Together with two holder no.8500700 the Universal band is a very secure Fixation.

Used material:

band, 2 Velcro strips
Cut the band at the right length for the patient.
Attach the two Velcro strips with the short side to the band.
Fix the band to the cannula with the two velcro strips.

Universal-Band for Endotrachealtubes and Tracheal cannulas


Universal-Band
ord.no..
8500890



Wires

Fixation on arm or leg

fix one of the Velcro strips on to the band, lay the wire down and close it with the Velcro strip. Close the band with the SECUTAPE pad.

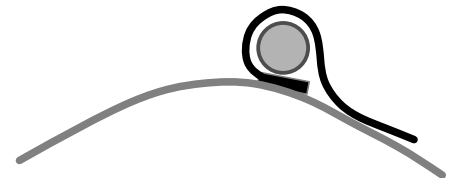

Most parts of the SECUTAPE® system can be combined with each other, Secutape fixations are reliable and offer the best possible hold on the skin. They can be used for several days.