



Application

Fixing set für Endotrachealtubes

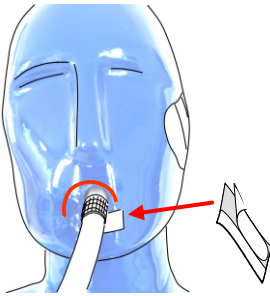


Pull the backing of the holder and stick them up to the cheekbones as shown on the pictures.

Take the cover of the Velcro pad off and wrap it overlapping around the cleaned and dry tube, dont cut it!

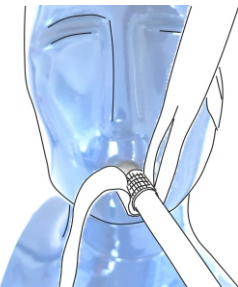

Fasten the band at the velcro pad and put one end through the slot. *see pic.*

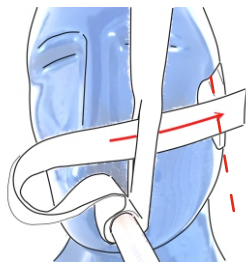

Fasten both ends on the holder, band **should always crosswise** the Velcro part of the holder. *see pic.*

- Hypoallergenic double sided Nylon band.
- The band is hydrophob and does not stretch.
- Two skin friendly fixingholder
- The fixation holds even if it gets wet!
- For patients with skin problems please use our holder from Hydrocolloid. Order no. 8500709

Fixing-set EXTRA

Fixing-set EXTRA
ord.no.
8500610

Fixing band EXTRA
ord.no.
8500810



All components are available separately.



Important:

the Velcrostrip on the tubus have to be covered with the band, otherwise there is a risk of skin problems.

Depending on the application, it lasts up to 4 days,

Most parts of the SECUTAPE® system can be combined with each other, Secutape fixations are reliable and offer the best possible hold on the skin. They can be used for several days.