


Application

Fixing-set für Endotrachealtubes

With hypoallergenic non stretching Nylon band.

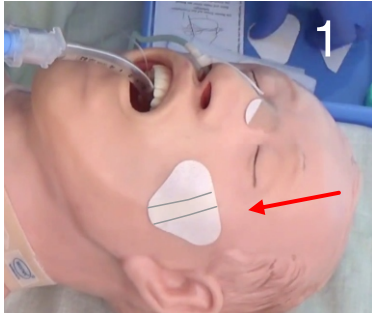

1

Cleanse the skin dry and free of grease.

Peel off the back of the holder.

Stick the holder onto the cheekbones.



2

Take the paper off the band and stick the band with the middle of the adhesive area to the tube.

Wrap it **twice** around the tube. *see pictures*



3

Fix both ends of the band on the Velcro area of the holder.

Important:

the band and the Holder are a cross!

To avoid possible skin damage turn the band like in the picture.

The new double-sided **Soft band does not need** to be twisted, it is soft all round.

- Hypoallergenic Nylon band, it is hydrohob and does not stretch.
- Skin friendly fixing holder
- The fixation holds even if it gets wet!

Depending on the application, it lasts up to 3 days, Hydroc. 6 days

Fixing-set

Fixing-set
ord.no.
8500600



Nonwoven-basic

Fixing-set
ord.no.
8500609



Hydrocolloid

Fixing band for set
ord.no.8500800



Fixing holder
ord.no.8500700




Nonwoven-basic

Fixing holder
ord.no.8500709



Hydrocolloid

Soft band for set
ord.no.8500805

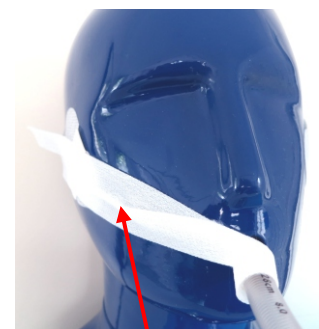

Suction and oral care:
Attach both ends on one side, so that the tube remains fixed and secured!

Most parts of the SECUTAPE® system can be combined with each other, Secutate fixations are reliable and offer the best possible hold on the skin. They can be used for several days.