


April 2024

# SECUTAPE®

SECUTAPE  
1986 - 2024  
**35**  
YEARS

## FIXATIONS

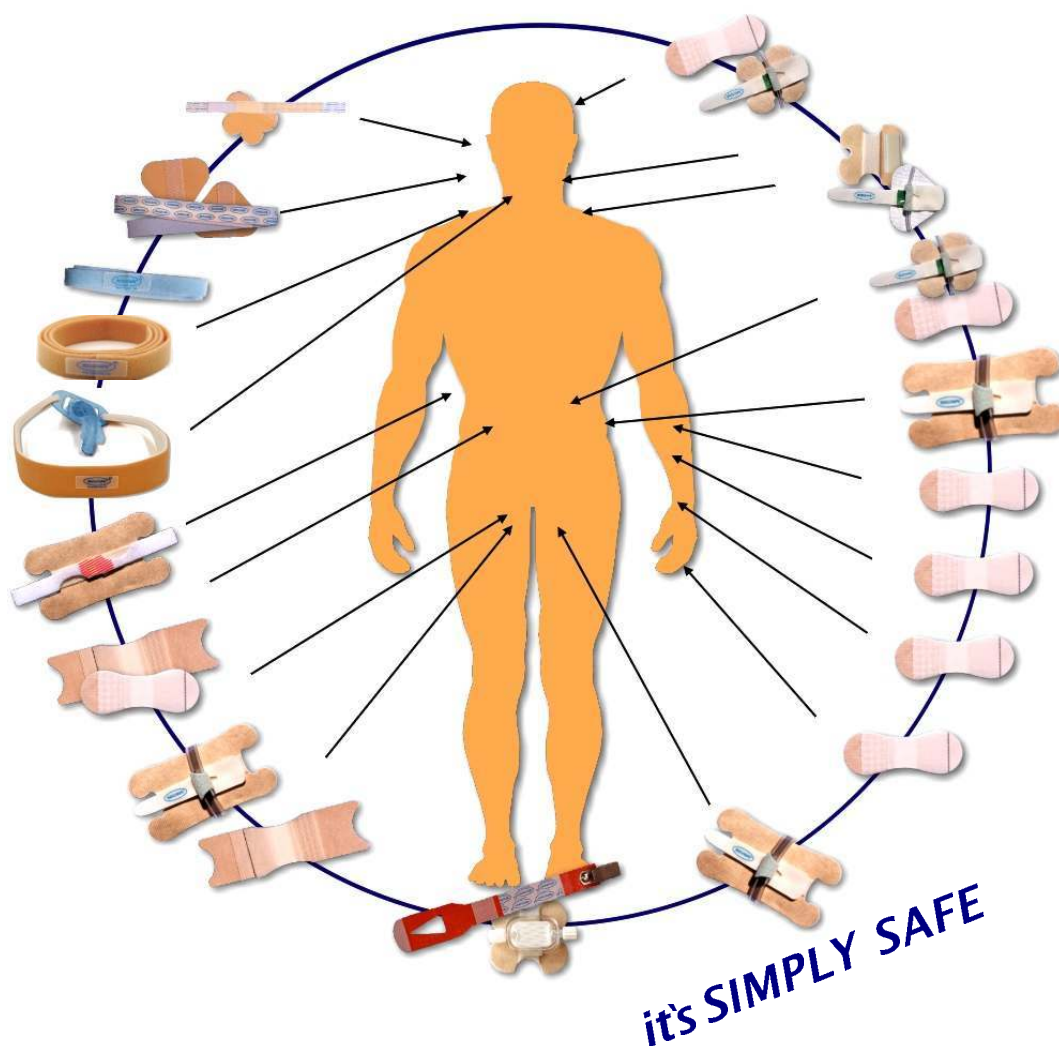

# CATALOG



**TechniMed AG**

SWITZERLAND since 1987

fon 0041 / 71 844 49 20

Kirchstrasse 54

CH-9400 Rorschach

fax 0041 / 71 845 52 45

mail: [info@technimed.ch](mailto:info@technimed.ch)

**Germany**

fon 0800 007 9892

fax 0800 941 3012

**Austria**

fon 01/ 440 7979

fax 01/ 440 7990

CE

Made in Germany

[www.secutape.ch](http://www.secutape.ch)

Copyright by TechniMed AG



## The SECUTAPE innovation:


The base plaster adheres completely to the skin; there are no detached spots!

The 'tube' is embedded in the adhesive and a velcro prevents the Flap from opening!

Via this Flap the movement of the tube is transferred to the whole plaster.

The fixation holds and can be opened and closed again, using the velcro flap.

All SECUTAPE fixations are based on the same basic idea, which was patented in 1986.

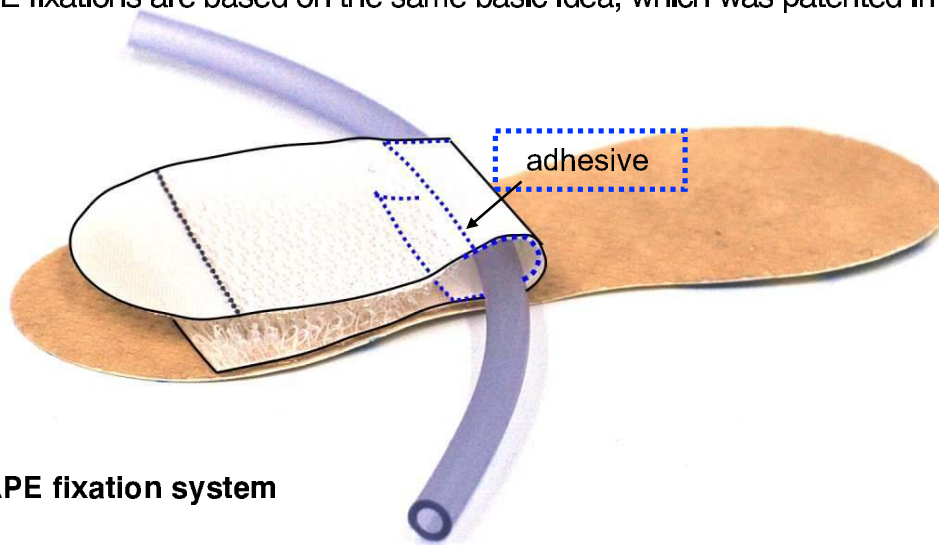

## The SECUTAPE fixation system

Provides safe and reliable solutions for practically all attachable catheters, drains, probes, tubes and cables; i.e. for all tube-shaped accessories.

We are the specialists in this field. For thirty years we have been manufacturing and distributing exclusively millions of fixations and we hold quite a number of patents.

We are offering our customers free training and advice through our field workers, including extensive documentation.

Our INFOCARE E-Learning platform for training of new employees and to refresh existing knowledge.

Our fixations are high-tech products that are manufactured in Germany. The materials used are manufactured specially for us and come from Switzerland, Germany, the USA and Italy. They are hypo-allergic, medically safe and contain no solvents.

Only through the use of high-quality materials in combination with our special and ergonomically designed shapes, we have achieved the astonishing holding power of our fixations.

Our products certainly come at a cost. But they hold better, do not cause skin damages and must be changed less frequently than others.

Calculated over the period of use, they are not expensive after all.



## Size 2 / 2+ / 3+

Standard fixation


**Since over 35 years our reliable fixations!**

for the fixation of :

- PEG
- CAPD
- CVC
- suprapubic catheters
- infusions
- urin-catheter
- redondrains
- ports
- wounddrains
- and all other tubes

**For heavy tubes pls. use size 3+**



The fixation can be opened and closed many times.

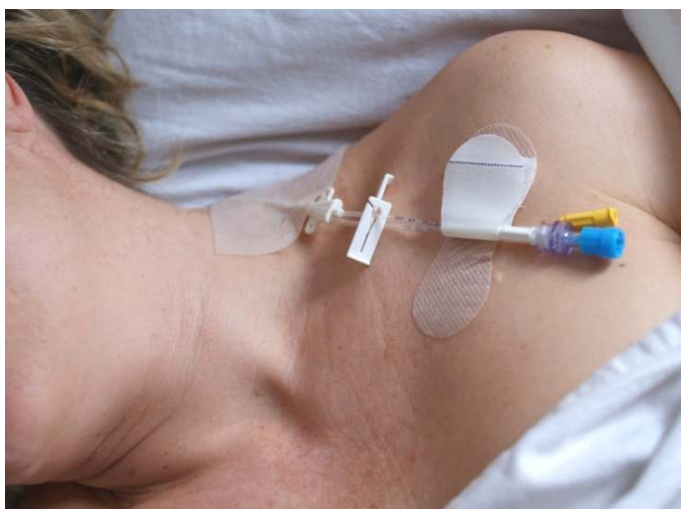

	<b>ord.no. 8500201</b>	1 PU 200 pcs.
	NW basic	
	singlepack	8 bags a.25pcs.
<b>2+</b>	<b>ord.no. 8500211</b>	1 PU 1000 pcs.
		20 bags. à 50 pcs.
		9X3.5 cm
		

	<b>ord.no. 8500301</b>	1 PU 200 pcs.
	NW basic	
	singlepack	8 bags a.25pcs.
<b>3+</b>	<b>ord.no. 8500311</b>	1 PU 1000 pcs.
		20 bags. à 50 pcs.
		15x4.5cm
		

**Stays appr. 3 - 5 days**

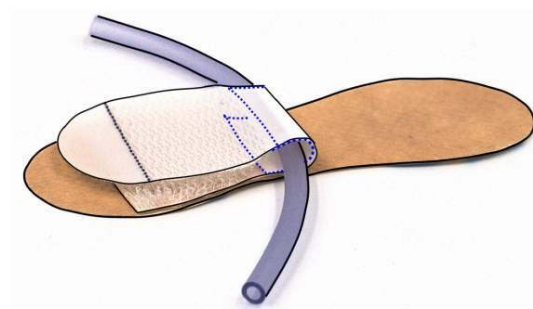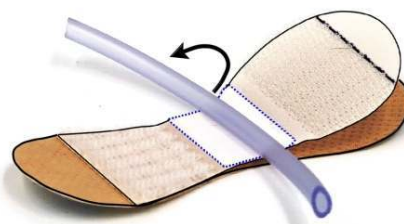

Made in Germany  
[www.secutape.ch](http://www.secutape.ch)

# SECUTAPE®

## Velcro-binder

### A new universal system for all applications and diameters!

Velcro binder holds reliably, **without any adhesive** tubes and wires with the new patented NANOPLAST® antislip foil.

The fixation can be opened and closed many times without leaving any marks on the tube!


Velcro binder comes as a set with different holders, also in Hydrocolloid for every application.

Due to the high quality breathing material, the fixation stays even when it gets wet.

NANOPLAST® has the so-called Gecko effect and is made out of silicon with aprox. 29000 nano legs per cm.

The fixation is very good for urine catheter and especially for the fixation of implanted tubes/lines.

### Velcro-binder

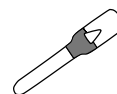


ord.no. 8501000 PU 200 pcs.  
singlepack. 8 bags a 25 pcs.



Velcro binder - Set SMALL

Hydrocolloid


ord.no. 8501100 PU 100 pcs.  
singlepack. 4 bags a 25 pcs.

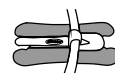

Velcro binder - Set BIG

ord.no. 8501130 PU 100 pcs.  
singlepack 4 bags a 25 pcs.

**Stays up to 3 days,  
Hydrocolloid up to 5 days**

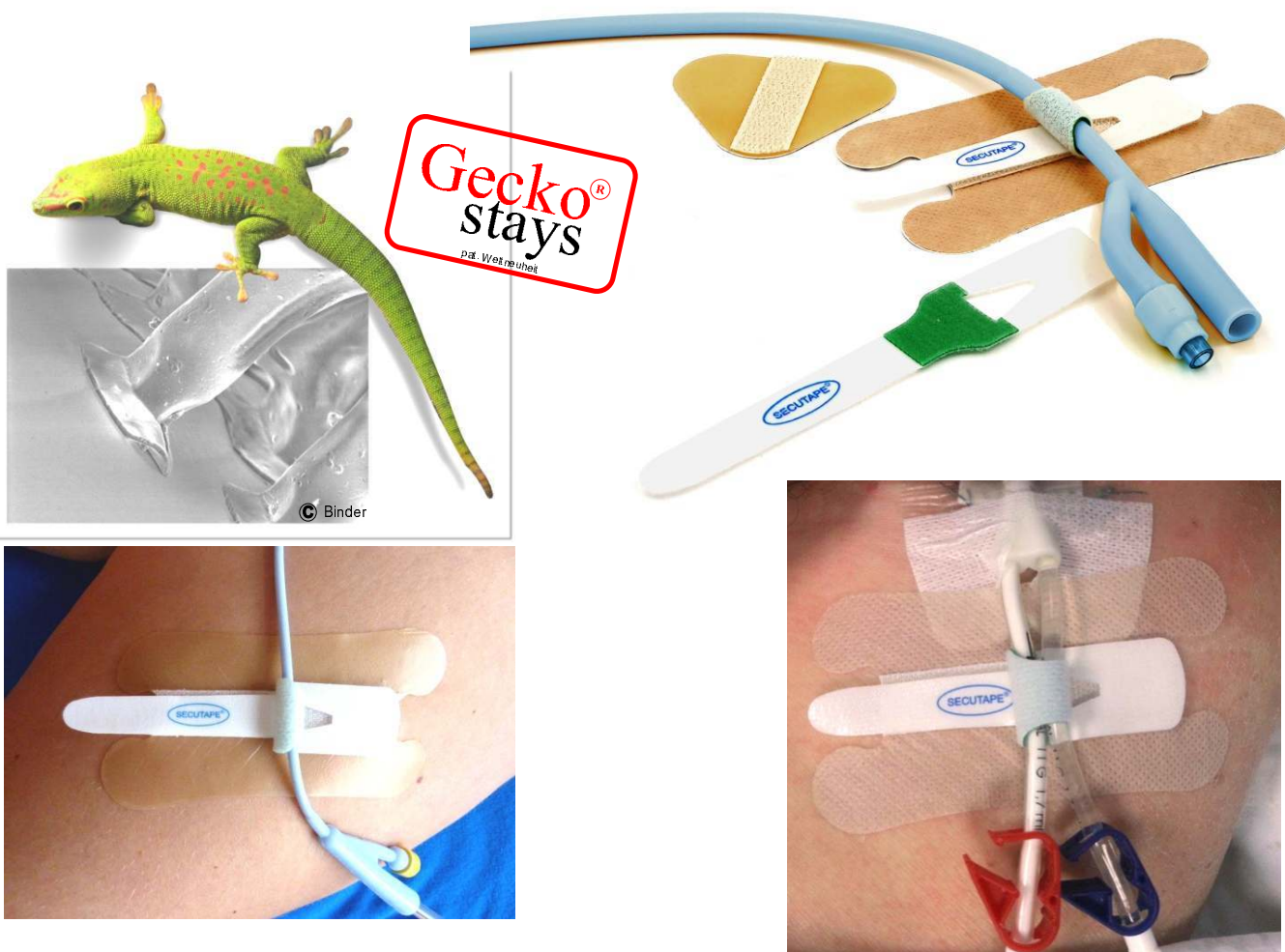

CE

Made in Germany  
[www.secutable.ch](http://www.secutable.ch)






Unifix

## Fixation for "tubes and wires" especially for CVC

UNIFIX is perfectly suited for the fixation of thinner catheters, Probes and lines, e.g. multi-lumen CVC...

The basis is a very good holding, ergonomic shaped holder made of skin-friendly, breathing tape that also holds when it gets wet.

The catheter is placed in an adhesive bed and held securely with a Velcro strap.

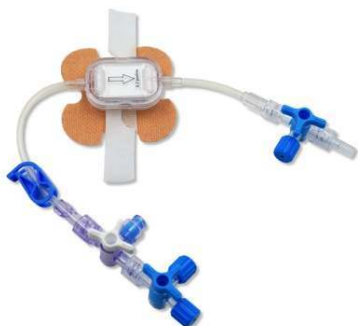

UNIFIX can also be used to attach filter, transducer or similar parts securely to the body.



**Unifix**

ord.no. 8501500 PU 200 pcs.



8 bags. à 25 pcs.  
singlepack

holder 6 x 6 cm



**Unifix Hydrocolloid**

ord. no. 8501509 PU 200 pcs.



8 bags. à 25 pcs.  
singlepack

holder 6.5 x 5 cm


As an additional fixation for the tubes a non-slip holder with a clamp.

**KROKOFIX small**

ord. no. 8501410



The use of two fixations reduces strain and movement on the CVC, which prevents infections.

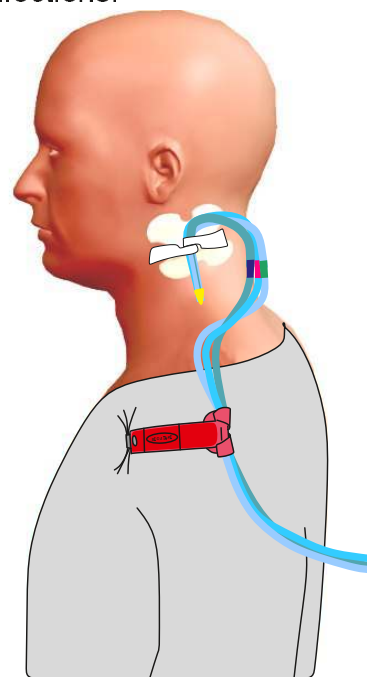

CE

Made in Germany

[www.secutape.ch](http://www.secutape.ch)



## Fixing set for big lumina

### Our best holding Fixation for large tubes!

Good for mobile patients!

#### Specially designed for:

chest tubes, ECMO, rinsing catheters and  
for the fixations of urinary catheters.

Through a special coating it even holds  
silicon catheters!

Due to the high quality nonwoven material,  
**the fixation stays even when it gets wet.**



**Stays appr. up to 4 days  
hydrocolloid up to 6 days.**



CE

Made in Germany  
[www.secutape.ch](http://www.secutape.ch)



## Nasal fixations

Fixation for nasale sondes

### Patented two piece fixation for all nasal sondes.

A specially shaped basic holder made of a skin friendly and breathable nonwoven or a hydrocolloid.

The basic holder has a Velcro part, a second strap with Velcro is glued to the tube.

The holder remains on the nose, if necessary only the straps are changed.

**There is no more skin damage** caused by the constant tearing of the plaster strips. The probe is hanging loose in the nose without causing any problem.



ord.no.. 8500400    **1 PU 200 pcs.**  
hydrocolloid    8 bags a.25pcs.  
singlepack.  
set  
holder 4.4 x 3.5cm  
strap 7.5 x 0.8 cm



ord.no.. 8500401    **1 PU 200 pcs.**  
nonwoven    8 bags a.25pcs.  
singlepack.  
set  
holder 4.4 x 3.5cm  
strap 7.5 x 0.8 cm



ord.no.. 8500410    **1 PU 400 pcs.**  
board à 4 pcs.  
singlepack.



ord.no.. 8500440    **1 PU 500 pcs.**  
board à 5 pcs.  
singlepack.

**Stays appr. 3 days  
hydrocolloid 5 days**



Made in Germany  
[www.secutape.ch](http://www.secutape.ch)





## Fixing set for endotracheal tubes oral/nasal

**Since over 30 years our reliable fixation for endotracheal tubes!**

Easy to fit by just one person and inexpensive due to longer application time..

Two holders of a very good adhering and skin friendly nonwoven or hydrocolloid with Velcro

Non stretchable, hypoallergenic nylon straps which do not absorb liquid.

The straps are centrally coated with a special adhesive.

**The system holds very well when used correctly and allows safe aspiration without a second person.**

The standard strap is **soft on one side**.

The Softband is **soft on both sides**, which prevents skin damages.



**ord.no. 8500609** 1 PU 100 pcs.  
**hydrocolloid** 4 bags. à 25 pcs.  
singlepack set

**ord.no. 8500600** 1 PU 200 pcs.  
nonwoven 8 bags. à 25 pcs.  
singlepack set

**ord.no. 8500709** 1 PU 200 pcs.  
**hydrocolloid** 8 bags. à 25 pcs.  
singlepack  
holder 6.5x5cm

**ord.no. 8500700** 1 PU 200 pcs.  
8 bags. à 25 pcs.  
singlepack  
holder 6.5x5cm

Fixing Strap standard  
**ord.no. 8500800** 1 PU 200 pcs.  
8 bags. à 25 pcs.  
strap 40x1.6 cm

**NEW** Softband  
**ord.no. 8500805** 1 PU 200 pcs.  
8 bags. à 25 pcs.  
singlepack  
strap 40x1.6 cm

**Stays appr. up to 3 days,  
hydrocolloid up to 5 days.**



Made in Germany  
[www.secutable.ch](http://www.secutable.ch)



# SECUTAPE®

## Tube fixing set EXTRA


**Our known tube fixing set with a newly developed strap.**

With our proven nonwoven holders and a very soft hypoallergenic nylon strap that does not absorb humidity and **does not stretch**.

The band has a slot in the center for a secure position of the tube.

The band is very well held on the tube with a self-adhesive Velcro strip.

The system holds very well when correctly installed and allows safe aspiration **without a second person**.

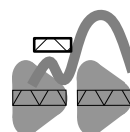


**Fixing set EXTRA**  
for endotracheal tubes

ord.no. 8500610 1 PU 200 pcs.

singlepack 8 bags à 25 pcs.

Both parts are available separately

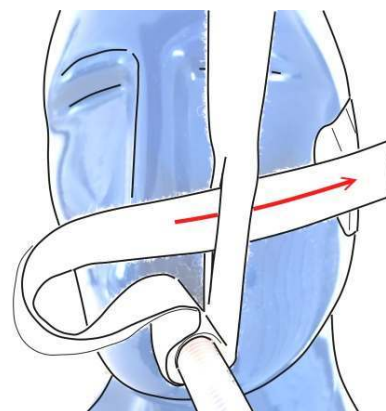

CE

Made in Germany

[www.secutape.ch](http://www.secutape.ch)



## Short-term fixing tape

self-adhesive  
for endotrachealtubes.

### Fixing tape for the short-term fixation of endotracheal tubes.

Newly developed high-quality self-adhesive fixing tape,  
specially designed for the anesthesia.

The hypoallergenic, skin-friendly and excellent  
holding tape does not stretch and rip.

It is packed in a hygienic dispenser and easy to handle  
due to its shape and the special liner.

The perforated dispenser is opened at the front, which  
eases the withdrawal and storage.



### Short-term fixing tape

for endotracheal tubes

**ord.no. 8500820** 1 PU 400 pcs.  
box a. 100 pcs.  
37x3cm



This strap does not replace  
a good long-term fixation!



Made in Switzerland  
[www.secutape.ch](http://www.secutape.ch)

# SECUTAPE®

## Tube-band


### Pat. fixation for endotracheal tubes

A hypoallergenic soft nylon band, **it does not stretch** and does not absorb any liquid.

Since the strap does not stretch no need to tight.

Due to the silicon coating on one side the strap has no hard edges, does not slip on the skin and the tube stays in place!

It is easy in handling, prevents most skin problems and must be changed less frequently than conventional textile or foam straps.

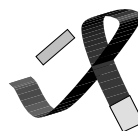

### Tube-band

ord.no. 8500850 1 PU 200 pcs.

singlepack. 8 bags à 25 pcs.

75 cm long



Stays approx. up to 2 days.



CE

Made in Germany

[www.secutable.ch](http://www.secutable.ch)



## Universal-band

### For endotracheal tubes & tracheal cannulas

Newly developed double-sided nylon band is hypoallergenic and pleasantly soft.

**It does not stretch** and does not absorb liquid. Therefore the patient does not have a humid band around the neck.

The band is also easy to clean.

It has to be changed less frequently than the usual foam or fabric bands.

Since the band should not be applied tight, there will be less skin damage and more comfort for the patient.

### One product for both applications!



Time of use approx. up to 3 days.



CE

Made in Germany

[www.secutape.ch](http://www.secutape.ch)





**Tracheofix**  
Comfort

## The advantage of new materials!


### Fixation band for tracheal cannulas

Newly developed thick and fluffy nylon band.

Hypoallergenic and pleasantly soft.

**The high quality band does not stretch and does not absorb liquid.**

It is easy to apply and due to its material it has to be changed less frequently than the usual foam or fabric bands.

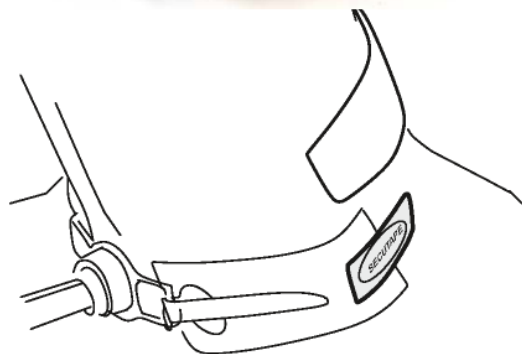

### Superabsorber Pad


In the case of skin damage and heavily sweating patients a superabsorber pad is available.

The med. grade absorber pad sticks by itself and keeps the skin dry.

It is therefore an ideal solution for long-term fixation.



### Tracheofix Comfort

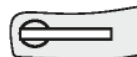

length 50cm x 3cm

ord.no. 8501750 PU 100 pcs.

singlepack. 4 bags. a 25 pcs.



### Superabsorber Pad

length 20 cm

ord.no. 8501770 PU 100 pcs.

singlepack. 4 bags. a 25 pcs.



**Stays up to 3 days  
including absorber pad  
up to 5 days.**

CE

Made in Germany  
[www.secutape.ch](http://www.secutape.ch)




# Krokofix



## Fixation for "tubes and wires" at clothes and bed linen.

- two functions!
- simple handling !
- two sizes with stainless steel clamp.
- secure and non slipping adjustment!
- variable use!
- disinfection with usual means!



A manual anti-slip fixation for tubes of all sizes.  
Good for mobile patients and strong pulling.

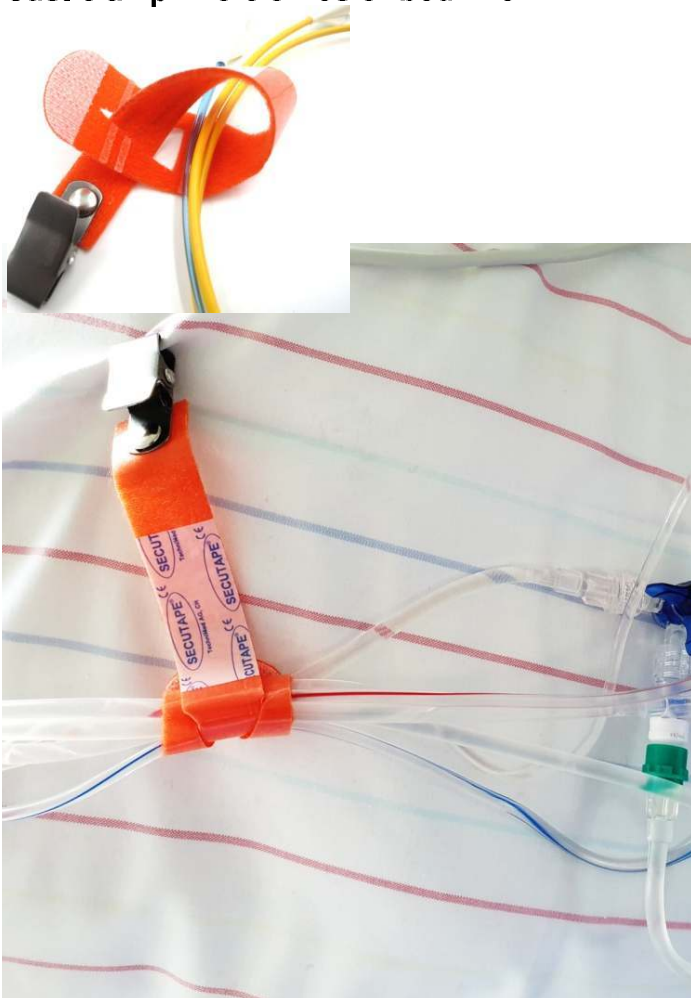
**Just clamp it to clothes or bed linen.**



	<b>Krokofix big</b>
ord.no. 8501420    1 PU 200 pcs.	
	
23 cm	

	<b>Krokofix small</b>
ord.no. 8501410    1 PU 200 pcs.	
	
16.5cm	

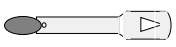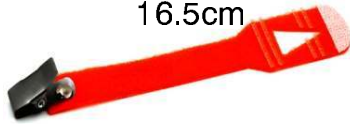

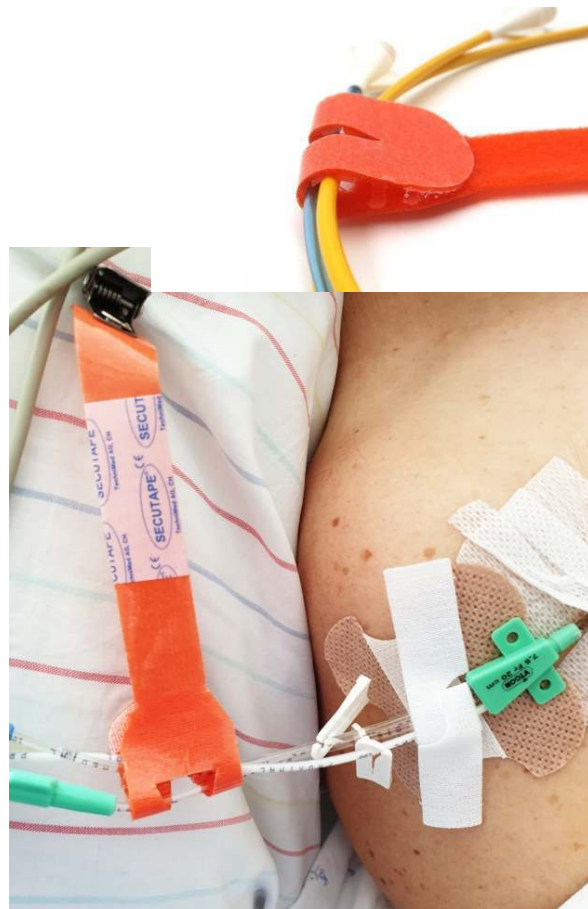CE

Made in Germany  
[www.secutape.ch](http://www.secutape.ch)

# SECUTAPE®

## Head holder-set ECMO

### Adjustable head holder set for fixation of an veno-arterial ECMO

The elastic head holder is a thick and soft nylon band and fits all head sizes.

The holder is, in combination with the Velcro straps and the Hydrocolloid holders, a very reliable Fixation, which gives the patient a certain room for movement.

The additional Fixation on the neck helps to reduce the strain on the hoses.

With the enclosed KROKOFIX, which should be fixed to the bed linen, to reduce the strain on the drains.



### Head holder-set: ECMO

ord.no. 8501610, PU 10 set




CE

Made in Germany  
[www.secutape.ch](http://www.secutape.ch)



**FIXIERUNG  
FIXATION  
FISSAGGIO**

*SIMPLY SAFE*

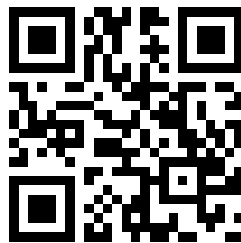

[www.secutape.ch](http://www.secutape.ch)