



Fixing system

Application

Fixing tape for the short-term fixation of endotracheal tubes.

Selfadhesive fixing tape, for the anesthesia.

The hypoallergenic, skin-friendly and excellent holding tape does not stretch and rip.

Open the box on the marked side to take a tape out..

Application

Cleanse the skin.

Pull the liner off on one side and stick it on one cheek of the patient.

Pull the middle liner off and wrap the tape on the marking twice around the tube.

Pull the last liner off and stick the tape on the other cheek.

Attention: do not use if skin is bad or damaged. Peel off slowly after use!



TechniMed AG
SWITZERLAND
since 1986

Kirchstrasse 54
9400 Rorschach
info@technimed.ch

Switzerland
fon 0041/71 844 49 20
fax 0041/71 845 52 45

Germany
fon 0800 007 98 92
fax 0800 941 30 12

Austria
fon 01/440 79 79
fax 01/440 79 90

CE
Made in Germany
©TechniMed AG

www.secutable.ch