


**Application**

**Tracheofix Comfort**


Fixation band for tracheal cannulas.



- 1 Attach the short band with the Velcro strip to the cannula.
- 2 Attach the longer band with the Velcro strip on the other side of the cannula.

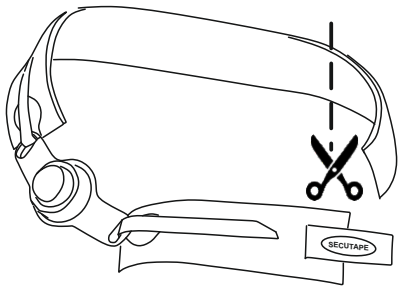

- 3 Lay the longer band around the neck and cut it off if its to long.

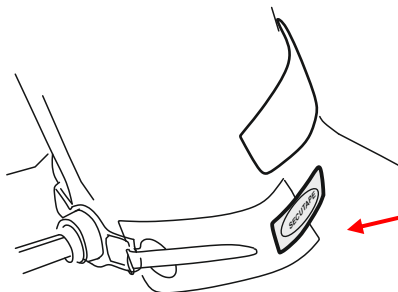

- 4 Close the Tracheofix with the SECUTAPE pad.  
**Dont tie to much the band does not stretch!**

A new thick and fluffy two-piece Nylon band, hydrophob and hypoallergenic. Therefore it does not stretch and does not get wet.

Tracheofix Comfort



Tracheofix Comfort  
ord.no.  
8501750



Depending on the application, it lasts up to 3 days

Most parts of the SECUTAPE® system can be combined with each other, Secutate fixations are reliable and offer the best possible hold on the skin. They can be used for several days.